

# BLOOD LANCETS

Used to prick the skin and gather blood samples of various tests

- Consistent depth penetration
- Fine gauge for less pain
- Easy-to-use
- Sterile, single use only
- Individually wrapped

- Ultra-sharp point tapers from all sides for minimum skin trauma and pain
- Can be used as splinter probe to remove shallow splinters

**LIVINGSTONE**



Item No.: **JPBLCT03**

Item No.	Description	Unit
JPBLCT03	Manual, Non-Retractable, Stainless Steel	200/Box
JPBLCT04	21G x 1.8mm, Stainless Steel Tip, Suits Autolet Glucolet Monolet Device	200/Box

**E.Z. BLOOD LANCETS**



Item No.: **JPBLCT04**

- Unique one-press activation with auto-retracting encased needle prevents accidental injuries
- Convenient, pre-loaded and ready to use

Item No.	Description	Unit
JPBLCT01	26G x 1.8mm, Device Free, Painless Low Flow, Yellow	50/Box

**E.Z. FLEXI**




Item No.: **JPBLCT01**

- Automatic lancing device
- Comes with 3 depth settings for absolute flexibility on all skin types or texture

Item No.	Description	Unit
JPBLCT02	23G x 1.3mm, 1.8mm, 2.3mm, Device Free	100/Box

#### INSTRUCTION FOR

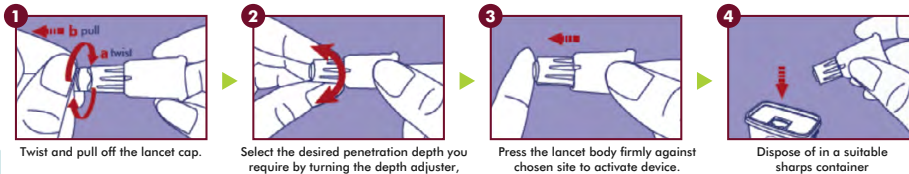

1 Twist and pull off the lancet cap.

2 Select the desired penetration depth you require by turning the depth adjuster, the lancet is pre-set 1.8mm. Depth settings; Low(1.3mm), Medium(1.8mm),

3 Press the lancet body firmly against chosen site to activate device.

4 Dispose of in a suitable sharps container

- Safe painless button activated
- Ergonomic design for comfortable grip
- Smaller lancet tip to make positioning on fingertip easier
- Bumps design on lancet tip for gentle pressure application around the puncture site
- Controlled needle depth for enough sample to suit most capillary blood requirements


**E.Z. LITE 3**



Item No.: **JPBLCT02**

Item No.	Description	Unit
JPBLCT06	26G x 1.8mm	100/Box
JPBLCT08	28G x 1.8mm	100/Box

#### HOW TO USE

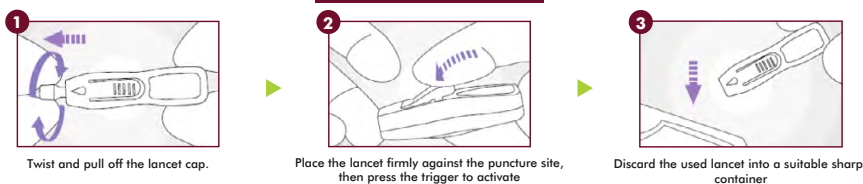

1 Twist and pull off the lancet cap.

2 Place the lancet firmly against the puncture site, then press the trigger to activate

3 Discard the used lancet into a suitable sharps container



Item No.: **JPBLCT06**

**PLEASE CONTACT YOUR ACCOUNT MANAGER TODAY TO PLACE YOUR ORDER**

**AUSTRALIAN TRUSTED TRADER**

**I ndis**  
Registered NDIS Provider

Follow us on **INSTAGRAM**  
@livingstoneinternational

**LIVINGSTONE**

