

klosure-strip®

Reinforced Wound Skin Closure Strips

Suitable for:

- Closure of lacerations and surgical incisions
- Can be used in conjunction with sutures/staples
- Supports wound after sutures/staples are removed

Contraindications


- Seek medical advice immediately for all serious wounds and burns
- Not suitable as a primary dressing
- Fragile skin or heavily aged skin
- Not for use on wounds with foreign material in them or active infection

Application:

Klosure strips can be applied directly to the skin to support wound closure. Start by positioning one end of the strip across the laceration. Anchor this end securely, then pull the strip tightly to create tension and carefully adhere it to the opposite side of the laceration. This will provide gentle, consistent pressure to help keep the incision closed and promote optimal healing.

Product	Item Number	Size
	LIVSTR01S	No. 01, 3 x 75mm
	LIVSTR02S	No. 02, 6 x 75mm
	LIVSTR03S	No. 03, 6 x 38mm
	LIVSTR04S	No. 04, 6 x 100mm
	LIVSTR05S	No. 05, 13 x 100mm
	LIVSTR06S	No. 06, 25 x 125mm

Ensure the strips are placed in the opposite direction of the wound to create effective alignment and tension

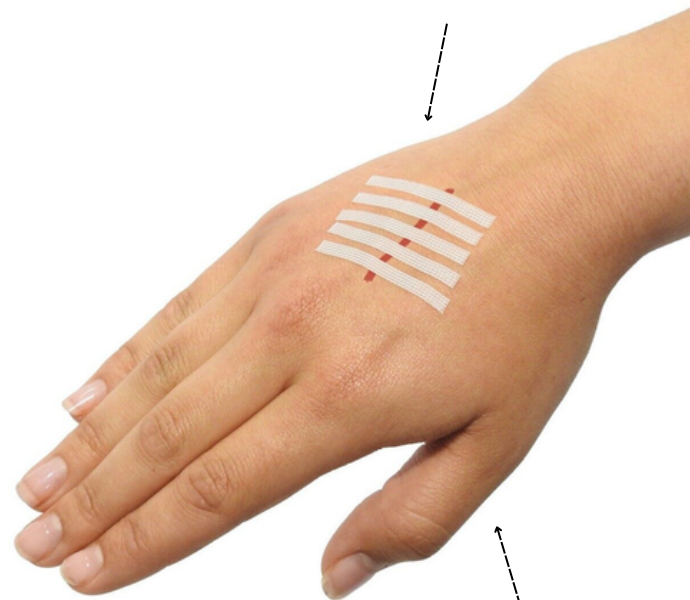

Remove by peeling the long edge off first to avoid re-opening the wound or soak with water before removal

BUY FROM OUR WEBSTORE
WWW.LIVINGSTONE.COM.AU
AND GET THE LATEST RANGE AND BEST PRICES



Follow us on INSTAGRAM
@livingstoneinternational

