

# FIX-A-ROLL

## Adhesive Dressing Fixation Retention Tape

### Suitable For:

- Suitable for all wound types as a secondary dressing
- Can be used to secure surgical devices



### Contraindications

- Seek medical advice immediately for all serious wounds and burns
- Not suitable as a primary dressing or for contact with the wound bed



### Application:

Fix-a-Roll can be applied directly to the skin and should not come into contact with the wound bed. It is ideal for securing non-adherent wound dressings or medical devices to the skin. To use, ensure the area around the wound is clean and dry, then apply the Fix-a-Roll over the non-adherent dressing, ensuring the edges are securely sealed to prevent any gaps around the dressing or insertion site.

Product	Item Number	Size
	FIXAROL2.5X10	2.5cm x 10m
	FIXAROL5X10	5cm x 10m
	FIXAROL10X10	10cm x 10m
	FIXAROL15X10	15cm x 10m



BUY FROM OUR WEBSTORE  
[WWW.LIVINGSTONE.COM.AU](http://WWW.LIVINGSTONE.COM.AU)  
 AND GET THE LATEST RANGE AND BEST PRICES



Follow us on **INSTAGRAM**  
[@livingstoneinternational](https://www.instagram.com/livingstoneinternational)

