



SCALPEL BLADE REMOVER

A *simply safer* way of
removing blades

Page 1 of 2

Enables safe removal, containment, and disposal of blades without the sharps injury risks of two-handed methods involving fingers, forceps, or re-sheathing



Item No.: **QLICKBLAD**

BladeFLASK Scalpel Blade Remover

- Single-handed blade removal
- Audible 'click' sound signifies safe scalpel blade removal and containment
- Built-in counter with automatic shutoff when maximum capacity is reached
- Mount on a wall, bench, or trolley

Unit: **Each**

BladeCASSETTE Scalpel Blade Remover

- Removes up to 3 scalpel blades
- Audible "click" sound signifies safe scalpel blade removal and containment
- Transparent cartridges
- Sterile

Unit: **100 Pieces/Box**



Item No.: **QLICKCASSETTE**



Item No.: **QLICKBRKT**

Universal Mounting Brackets

- Reusable
- Comes with screws and industrial strength adhesive
- Installation guide included
- Compatible with both the BladeFLASK and BladeFlask EVO

Unit: **Each**

BUY FROM OUR WEBSTORE WWW.LIVINGSTONE.COM.AU
AND GET THE LATEST RANGE AND BEST PRICES



Follow us on **INSTAGRAM**
[@livingstoneinternational](https://www.instagram.com/livingstoneinternational)





SCALPEL BLADE REMOVER

A *simply safer* way of removing blades

Page 2 of 2

Enables safe removal, containment, and disposal of blades without the sharps injury risks of two-handed methods involving fingers, forceps, or re-sheathing

Instruction for Use

PARTS OF YOUR BladeFLASK:

INDICATOR: SHOWS WHICH WAY SHARP EDGE MUST FACE.

THIS PART OF THE SCALPEL HANDLE IS CALLED THE TANG.

COUNTER

LANDING PAD

MOUNTING BRACKET

MOUNT TO WALL OR TROLLEY:

- If you haven't mounted your BladeFLASK before, prepare mounting bracket*.
A BRACKET HAS DOUBLE SIDED TAPE ON BOTTOM.
B OR USE SCREWS.
* See fitting template and instructions inside the bracket packaging.
- BladeFLASK can be mounted on a wall or on a trolley or bench.
SLIDE DOWN INTO BRACKET
SLIDE BACK IN TO BRACKET

NOTE: BladeFLASK is mounted on Mounting Bracket during use.

REMOVE BLADE SINGLE HANDED ACTION:

- Place blade on landing pad with the sharp edge facing to the right (or 'tang' up).
- Push scalpel handle into BladeFLASK until you hear a click.
CLICK!
- Remove handle.

NOTE: BladeFLASK is disposed of as medical waste after 100 blades have been inserted.

Blade Removers Comparison

	Single-handed	Automatic safety mechanism	Fixes device to wall or bench with mounting bracket	Certified sharps container	Blade directed away from user when removing	Indicates remaining uses with counter	Shuts off automatically when maximum capacity reached	Contains blade (from removal to disposal)	Compliant with international sharps safety + disposal standards	Removes large mortuary blades	Removes blades from circular handles
 BladeFLASK	✓	✓	✓	✓	✓	✓	✓	✓	✓		
 BladeFLASK EVO	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓

BUY FROM OUR WEBSTORE WWW.LIVINGSTONE.COM.AU
AND GET THE LATEST RANGE AND BEST PRICES



Follow us on **INSTAGRAM**
[@livingstoneinternational](https://www.instagram.com/livingstoneinternational)

